




Single Breast Exam

LF00970U & LF00977U

INSTRUCTION CARD



5-Year Warranty

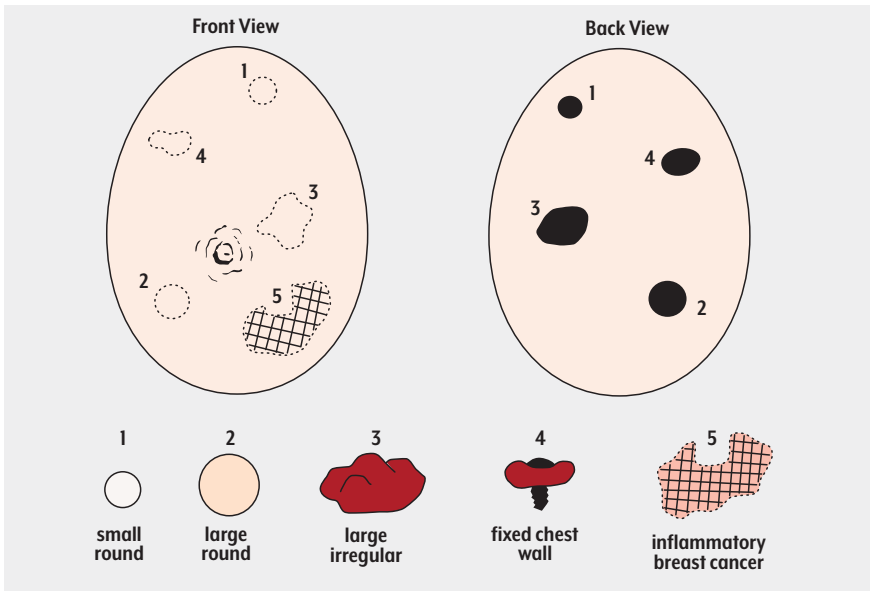

1. Insert the **SOFT PINK** or **BROWN** lumps into the respective cavities to represent **NORMAL** tissue.
2. Insert the **MEDIUM WHITE** or **HARD RED** lumps to represent benign cysts or cancerous lesions.
3. Plug the **RED CHEST WALL** lump into the hole in the base to represent a tumor that has grown onto the chest wall.
4. A light dusting of baby powder will give the trainer a more lifelike feel.
5. The trainer may be washed in soap and warm water if it becomes soiled.
6. The simulated breast tissue may leave an oily residue when in contact with cloth, paper, or other surfaces. Store the trainer on its base in the plastic bag provided.

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

Other Available **Life/form** Simulators

- LF00971U** Breast Self-Examination TearPad™
- LF00972U** Breast Self-Examination Reminder Stickers
- LF00973U** Breast Self-Examination Poster
- LF00980U** Advanced Breast Exam Simulator
- LF00984U** Breast Examination Simulator
- LF01143U** Testicular Exam Simulator



ONLINE:
EMAIL:
PHONE:
ADDRESS:

NascoHealthcare.com
lifeform@enasco.com
1.800.431.4310
901 Janesville Ave.
Fort Atkinson, WI 53538