


# Nasco Life/form® Single Breast Exam

## LF00970U & LF00977U Instruction Card




1. Insert the **SOFT PINK** or **BROWN** lumps into the respective cavities to represent **NORMAL** tissue.
2. Insert the **MEDIUM WHITE** or **HARD RED RED** lumps to represent benign cysts or cancerous lesions.
3. Plug the **RED CHEST WALL** lump into the hole in the base to represent a tumor that has grown onto the chest wall.
4. A light dusting of baby powder will give the trainer a more lifelike feel.
5. The trainer may be washed in soap and warm water if it becomes soiled.
6. The simulated breast tissue may leave an oily residue when in contact with cloth, paper, or other surfaces. Store the trainer on its base in the plastic bag provided.

Front View

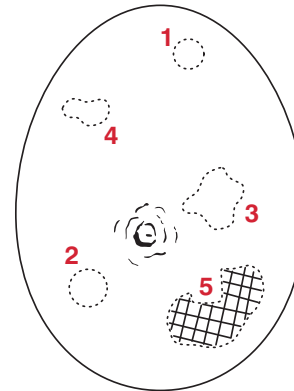

Back View

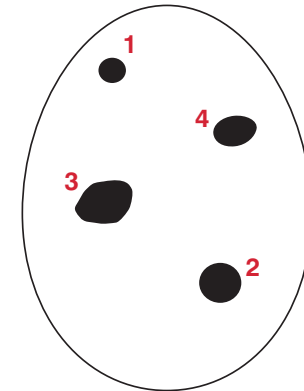

small round



large round



large irregular



fixed chest wall



inflammatory breast cancer



901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901

eNasco.com  
1-800-558-9595

E-mail: [lifeform@eNasco.com](mailto:lifeform@eNasco.com)

COPYRIGHT © NASCO 2013 PRINTED IN U.S.A. NP 035-12/RV10-13

## Other Available *Life/form*® Simulators

**LF00971U** Breast Self-Examination TearPad™

**LF00972U** Breast Self-Examination Reminder Stickers

**LF00973U** Breast Self-Examination Poster

**LF00980U** Advanced Breast Exam Simulator

**LF00984U** Breast Examination Simulator

**LF01143U** Testicular Exam Simulator



**LF00971U**



**LF00972U**



**LF00973U**



**LF00980U**



**LF01143U**

**LF00984U**

Actual product may vary slightly from photo.  
Nasco reserves the right to change product color, materials, or function as needed.