


# **KIM, KEVIN, AND KYLE**

## **Instructions for Use**

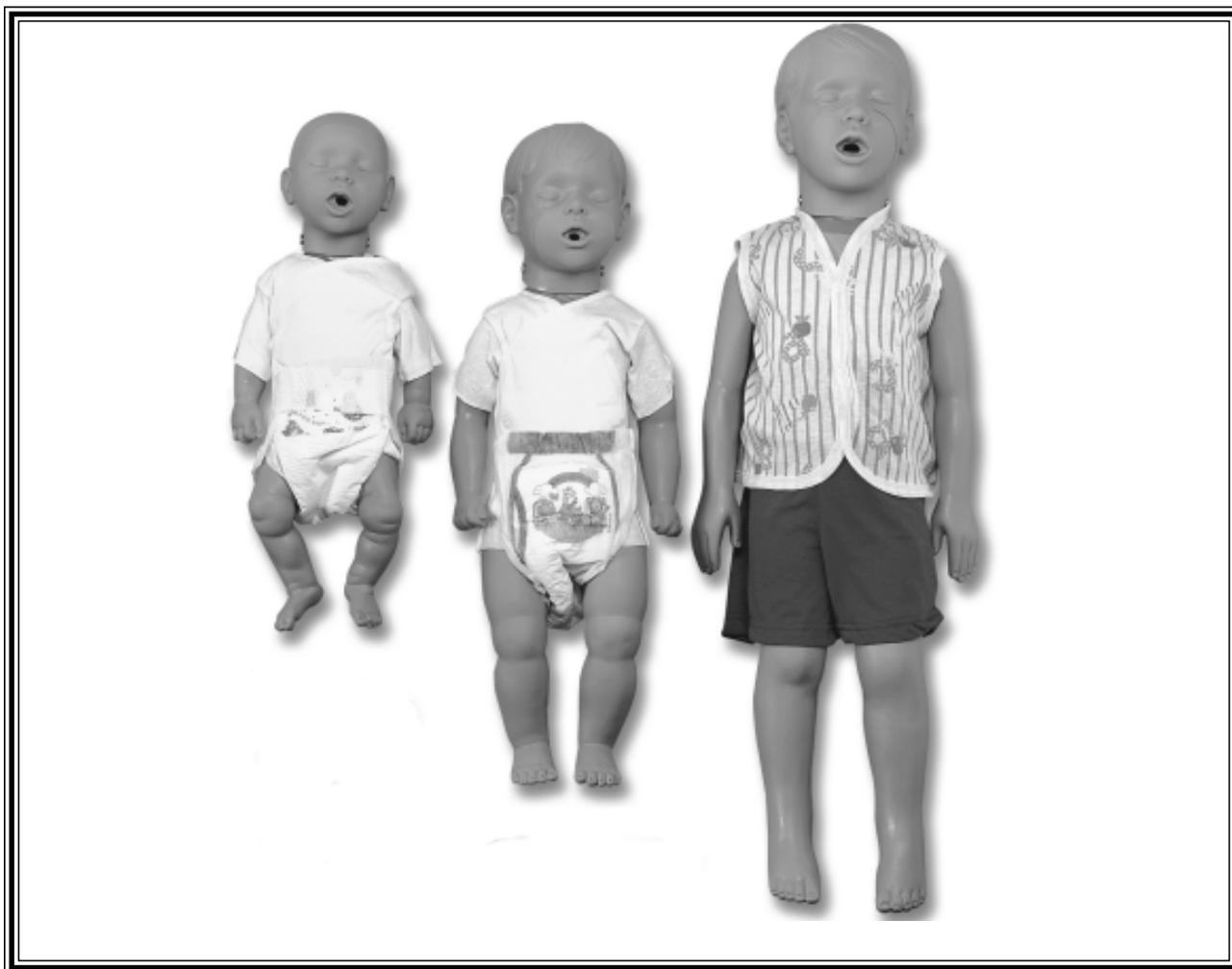

 **Simulaids**


The purpose of this instructional is to familiarize the user with the function, care, and maintenance of the Simulaids' Kyle, Kevin, and Kim CPR training manikins. It is not meant to provide instruction in Cardiopulmonary Resuscitation (CPR) or other emergency life support procedures. Please read it thoroughly before attempting to use your manikin.

### **Airway Removal**

1. Unsnap chest overlay from manikin.
2. Pull back retaining pin located at the base of the neck (fig.1) and lift the head off the body.
3. Remove mouth/ nosepiece from head.
4. Place hand inside head and grasp one-way valve. With other hand twist and pull flanged valve from the oropharyngeal opening in face (fig.2).
5. Remove and discard airway/lung assembly and flanged valve.

### **Airway Installation**

1. Insert the non-re-breathing valve through back of the face cavity and press (a slight twist when pressing assists placement) the flanged valve (flange goes toward mouthpiece) onto the non-re-breathing stem. Replace the mouthpiece.
2. Loop the tubing inside of head for both the Kim and Kevin airway installation (fig. 3).
3. Loop the lung over the front of the head pin. Position the elbow joint so that it points straight down. Pull the retaining pin back (fig. 1) and attach the head to the body.
4. With the manikin head slightly tilted back, lay the lung flat against the chest and snap the overlay in place.

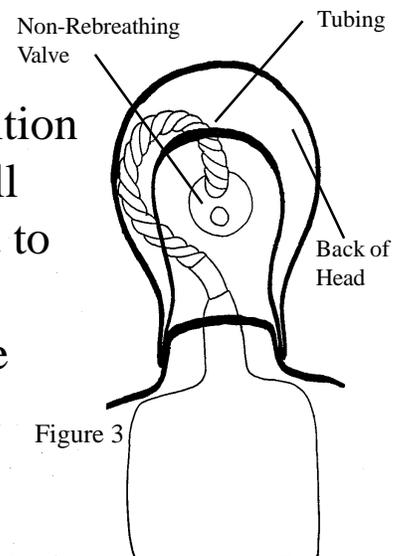

## **CLEANING & DISINFECTING**

**The only components that require scrupulous disinfecting are the mouth nosepieces.**

**The following procedure should be used:**

1. Clean in mild soapy water.
2. Rinse thoroughly in clear water.
3. Disinfect as per C.D.C. protocols.

### **SHIPPING WEIGHTS & DIMENSIONS:**

Kyle	9 lbs. 34"x11"x 7"
Kevin	5 lbs. 26" x 8" x 8"
Kim	4 lbs. 26" x 8" x 8"

### **REPLACEMENT COMPONENTS:**

<u>Part #</u>	<u>Description</u>	
2955	Kyle Airway System	24 pk
2905	Kim, Kevin Airway System	24 pk
2082	Kyle Mouth/ Nose Pcs.	10 pk
2162	Kevin Mouth/ Nose Pcs.	10 pk
2068	Kim Mouth/ Nose Pcs.	10 pk
2083	Kyle African/American Mouth/ Nose Pcs.	10 pk
2162B	Kevin African/American Mouth/ Nose Pcs.	10 pk
2069	Kim African/American Mouth/ Nose Pcs.	10 pk
2252	Kevin/Kim Carry Bag	
2260	Kyle Carry Bag	

## RETURN GOODS POLICY

Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain a RETURN AUTHORIZATION NUMBER. Merchandise cannot be accepted without prior return authorization. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture or distribute, contact our Customer Service Department.

## WARRANTY

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



PO Box 1289-16 Simulaids Dr  
Saugerties NY 12477

Toll Free: (800)-431-4310 Fax: (845)-679-8996

[www.simulaids.com](http://www.simulaids.com)