


# Detorsion with „one rope“ method

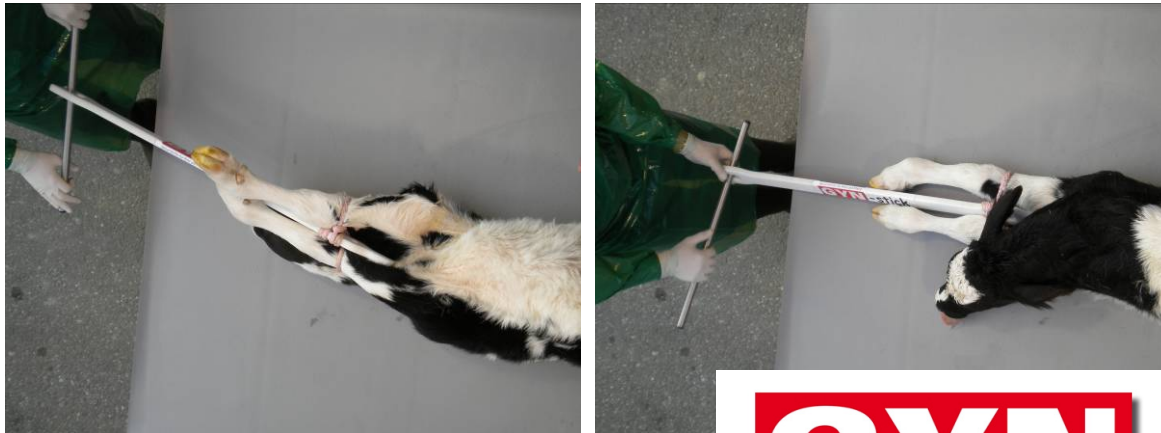

Exclusively distributed by:




**QUIDEE GmbH**  
Hauptstr. 53 A  
35315 Homberg  
Tel./Fax: +49(0)6633/9110857  
E-Mail: [info@quidee.de](mailto:info@quidee.de)  
Homepage: [www.Quidee.de](http://www.Quidee.de)

**GYN-stick®**  
Geburtshilfe für Rind und Pferd nach Dr. Schlederer  
...makes life easy!

## Detorsion „two-ropes“-/two persons-method



Put on ropes, fix them to the GYNstick extravaginally; tighten the GYNstick, it will automatically approach the calfs legs, Fix the ropes outside on the distal hole of the GYNstick, ready to turn=action.

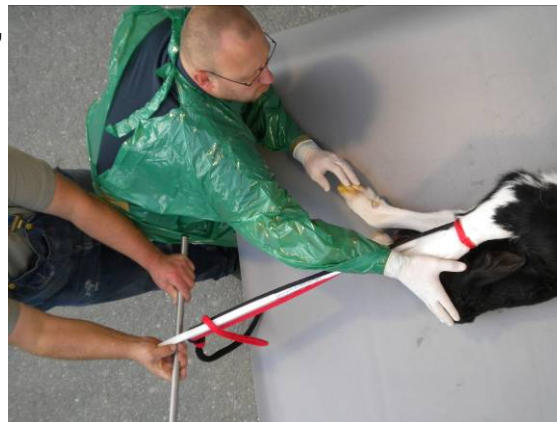

Insert the metal bar, while vet is monitoring „intracorporally“ the process and farmer is „turning“ the wheel. Both persons have enough space to act in contrast to common obstetric devices, which are simply too short; GYNstick is also a very safe instrument to avoid ruptures of the uterus or broken legs of the calf.

## Correction of displaced extremities (Kühn`s obstetrical crutch)

**GYN**-stick<sup>®</sup>  
Geburtshilfe für Rind und Pferd nach Dr. Schlederer



Turn the fine rope with the metal spoon around the proximal end of the displaced leg. Close the circle extravaginally on the end of the GYNstick; direct this loop in distal direction of the leg in order to get closer to the assistant and for further actions serving the calving process.

